



HBsAg Card Test

For the detection of Hepatitis B surface antigen in human whole blood / serum / plasma

One-step Hepatitis B Card Test for Whole blood / Serum / Plasma



- Clear background.
- Self contained test procedure.
- Specifically engineered for whole blood specimen, serum & plasma.
- 100% sensitivity and 100% specificity.
- Storage temperature 4°C 30°C.
- Detects all subtypes of HBsAg.
- Long shelf life up to 24 months.

TEST PROCEDURE

Add 2 drops of whole

blood or serum or plasma

to the specimen well.

BENEFITS

- Unmistakable results at 20 minutes.
- Simple and easy to use, does not require advanced medical training.
- Cost effective, can be used effectively in resource limited conditions.
- Accurate and reliable.
- Does not require cold chain & convenient for small volume users.

(3)

Result at 20 min.


Do not interpret after 25 min.


Ideal for laboratory / field conditions.





RESULTS INTERPRETATION

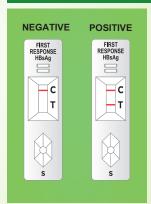

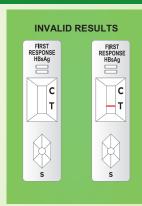

Pack Size

Specimen

transfer

device

KIT CONTENTS

- · Test device with desiccant
- · Assay buffer bottle
- · Specimen transfer device
- · Sterile lancet
- Alcohol swab
- Instructions for use

Catalogue No.

PI10FRC25	25 Tests / Kit
PI10FRC30	30 Tests / Kit
PI10FRC25CE	25 Tests / Kit
PI10FRC30CE	30 Tests / Kit

ORDERING INFORMATION info@premiermedcorp.com

Type: Bulk Packing

EXTERNAL LABORATORY EVALUATION REPORTS

Add 1 drop of assay

buffer to the

specimen well.

Place of Evaluation	Year of Testing	Sensitivity	Specificity
Biosynex, France	2018	100%	100%
Virus Research Institute, Uganda	2018	99.1%	100%



Manufactured by **Premier Medical Corporation Private Limited**

A1-302, GIDC, Sarigam 396155. Dist. Valsad, Gujarat, INDIA • Customer support email : info@premiermedcorp.com

• Tel.: +91 260 2780112/113, • www.premiermedcorp.com

Distributed By:



PT Sangkala Inter Medika

Komplek Kopo Mas Regency Blok 8 Kaveling No O Kel Margasuka Kec. Babakan Ciparay Kota Bandung Jawa Barat Indonesia WhatsApp: +62 812-2240-6748