



The Clear Choice...

One Step Malaria Ag. *P.f.* and *PAN* Combo Detection Card Test


Detect Malaria *P.f.* and *PAN* infection on single rapid card test (Whole blood)

FEATURES


- ▶ Detects all 4 species: *P. falciparum*, *P. vivax*, *P. ovale* and *P. malariae*.
- ▶ Provides immediate double check for *P. falciparum*.
- ▶ Self contained test procedure
- ▶ Detects as low as 200 parasites/ μ l.
- ▶ Test at any time of the day
- ▶ Long shelf life upto 24 Months.
- ▶ Storage temperature 1°C - 40°C.

BENEFITS

- ▶ Unmistakable results at 20 minutes.
- ▶ Differential diagnosis between *P. falciparum* and other Plasmodium species.
- ▶ Ideal for field testing
- ▶ Simple and easy to use.
- ▶ Cost effective
- ▶ Ideal in resource-limited condition
- ▶ Ideal for detecting low parasitemia levels.
- ▶ Allows immediate testing of specimen.

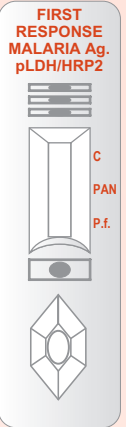

TEST PROCEDURE

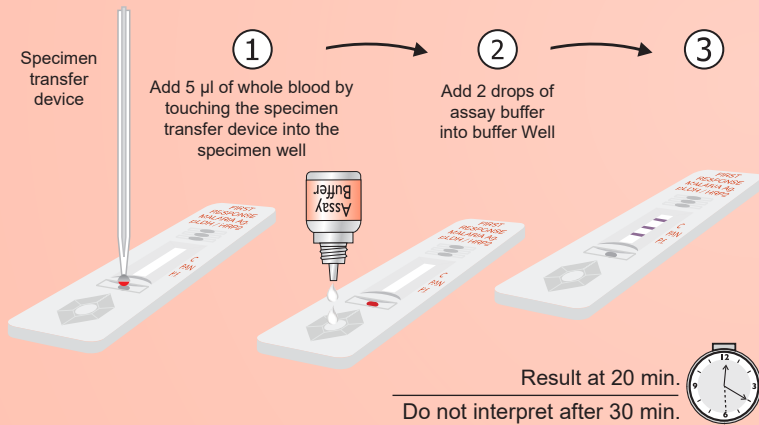


KIT CONTENTS

- Test device with desiccant
- Sterile lancet
- Assay buffer bottle
- Alcohol swab
- Specimen transfer device
- Instructions for use


EXTERNAL EVALUATION DATA

Place of Evaluation	Year	Sensitivity				Specificity	
		PAN	P.f.	PAN	P.f.	PAN	P.f.
Ghana	2015	96%	100%	100%	100%	100%	100%
WHO Evaluation Round 6	2015	Panel detection score				Specificity	
		200 p/ μ l	2000 p/ μ l	200 p/ μ l	2000 p/ μ l	-	-
		91.4%	100%	82%	100%	100%	100%

RESULTS INTERPRETATION

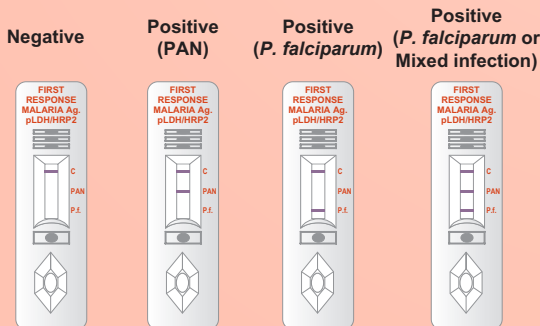

Invalid Results

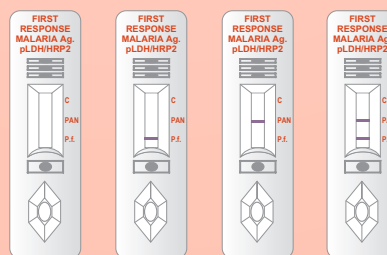

ORDERING INFORMATION

Info@premiermedcorp.com

Type : Bulk Packing

Catalogue No.	Pack Size
PI16FRC05	05 Tests / Kit
PI16FRC25	25 Tests / Kit
PI16FRC05CE	05 Tests / Kit
PI16FRC25CE	25 Tests / Kit

Type : Single Packing

PI16FRC10s	10 Single tests / Kit
PI16FRC25s	25 Single tests / Kit

Distributed by :



PT Sangkala Inter Medika

Komplek Kopo Mas Regency Blok 8 Kaveling No 0
Kel Margasuka Kec. Babakan Ciparay Kota Bandung
Jawa Barat - Indonesia
WhatsApp : +62 812-2240-6748